

NUSTAT TACTICAL HEMOSTATIC DRESSING



NuStat® Tactical rapidly controls bleeding allowing for quick stabilization and transport of the patient.

WHAT IS NUSTAT® TACTICAL ?

NuStat® Tactical is a blend of highly effective, non-impregnated, cellulose and silica based hemostatic fibers. The cellulose promotes the contact activation pathway of the coagulation cascade by absorbing blood fluids, resulting in the localized concentration of platelets and clotting factors. The silica-based fibers simulate the negative ions secreted by activated platelets, which further influence the coagulation cascade.

HOW IS NUSTAT® TACTICAL DIFFERENT?

NuStat® Tactical's dual hemostatic fibers do not contain coatings or biologics to activate the coagulation cascade. NuStat Tactical requires no preparation, is durable, absorbent and easy to apply. NuStat Tactical does not lose its hemostatic mechanism in the presence of bleeding. NuStat Tactical contains no latex or animal proteins.

HOW IS NUSTAT® TACTICAL PACKAGED?

NuStat® Tactical is vacuum packed and z-folded in a compact, easy to open, durable, waterproof pouch.

ORDER INFORMATION

Size	Dressing per box	Reference
3'' X 24''	10	5-0324

Bâtiment Le Diaph - 13 rue François Le Toullec - 56100 Lorient

Tél : +33 (0)2 97 80 10 40 - www.pc-equipment.fr



INSTRUCTIONS FOR USE:

1. Open package & remove NuStat® Tactical. Do Not Cut.
2. Apply NuStat® Tactical directly to the inner surface of the wound.
3. Apply firm pressure so NuStat® Tactical makes direct contact with the bleeding area.
4. Maintain pressure. If bleeding continues, apply new NuStat® Tactical as directed above.

INDICATION STATEMENT:

NuStat® Tactical is a single-use hemostatic wound dressing applied externally with mechanical compression to temporarily control bleeding in lacerations, punctures, abrasions and incisions.

Bâtiment Le Diaph - 13 rue François Le Toullec - 56100 Lorient

Tél : +33 (0)2 97 80 10 40 - www.pc-equipment.fr

